



CHALLENGE SERIES | Zygomatic™

DRILLING SEQUENCE FOR ZYGOMATIC

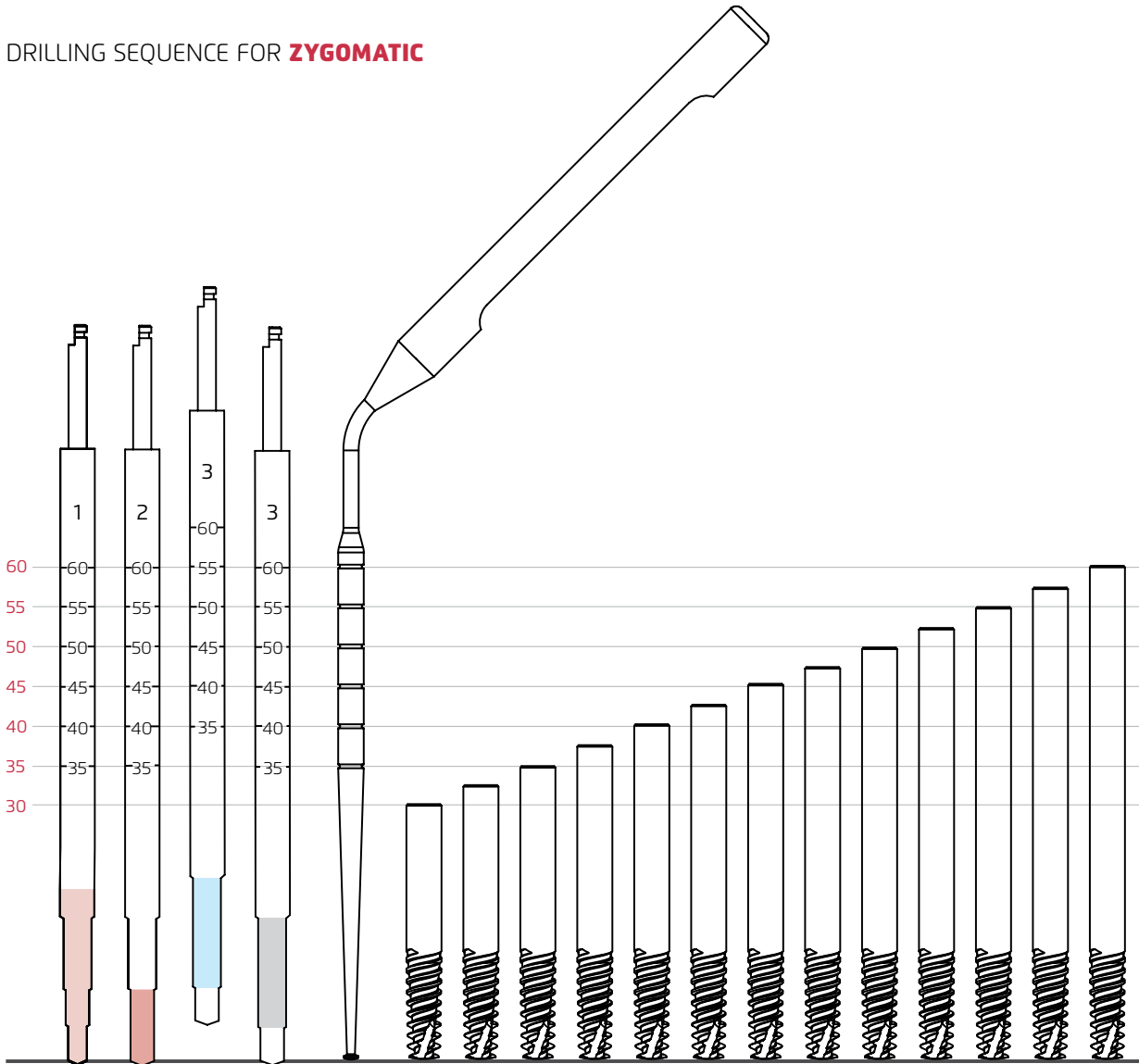

RECOMMENDED STEP DRILL PROTOCOL

Drill Diameter [mm] / Drill No.		Ø2	1	2	3
Drill Speed [RPM]		900-1200	700-900	500-700	400-600
Bone Type					
IMPLANT DIAMETER	D4	↓	↓		
	D3	↓	↓	↓	
	D2	↓	↓	↓	↓ -5mm
	D1	↓	↓	↓	↓

 Drill osteotomy to implant

 -5mm
 Drill 5mm Less

Prior to the use of Zygomatic implants in the Zygoma region, additional training is recommended.